

Deltoid Intramuscular Injection Trainer (L-DIM-01)

-Realistic and soft skin, no needle marking after injection

-500+ stabs per module with 22-23 gauge needles

-Wearable on person and manikin (strap)

-Able to inject with liquid and have drainage hole

-Able to palpate for anatomical important landmarks

-Instruction and Maintenance;

- Tap water only should be used for injecting
- There is a drainage hole for removing the water out. Helps the water flow out quickly by positioning them vertically or let them dry naturally
- After using can be wipe with clean wet cloth; Do not use alcohol for wiping
- Do not write or scribble on the model
- Silicone material are high tolerance to extreme temperature (-40°C to +200 °C).
- Recommend to store at room temperature

- Life Size Arm Model x 1ea (255mm*55mm*60mm / appx. 475.5g)

